

LMA[®] MADgic[®]

Laryngo-Tracheal Mucosal Atomization Device

Introducing the LMA[®] MADgic[®] Laryngo-Tracheal Mucosal Atomization Device—A versatile choice to administer medication across the entire upper airway and beyond the vocal cords.

For all of your difficult and awake intubations, bronchoscopies, transesophageal echocardiograms and endoscopies, the MADgic[®] Device will have you and your patients breathing easier with innovative features that are easy to swallow.

The MADgic[®] Device is not prefilled, so you can select the type of medication and exact dosing desired. And with its malleable stylet, you can attain precise, targeted delivery specific to your patient's anatomy.

THOROUGH, TARGETED ANESTHETIC COVERAGE

- Atomized particles are gently dispersed across a broad area of the mucosa for optimal coverage
- Greater patient comfort—less stressful intubation/extubation
- Less risk of patient injury

NARROW PROFILE


- Designed to fit through vocal cords, down LMA[®] laryngeal mask or into nasal cavity

SAFER


- Single use prevents cross-contamination
- Exact dosing reduces risk of toxicity
- Not made with natural rubber latex

MEDICATION SPECIFIC


- Use topical anesthetic, vasoconstrictor, thrombin, etc.
- Refill as needed on same patient

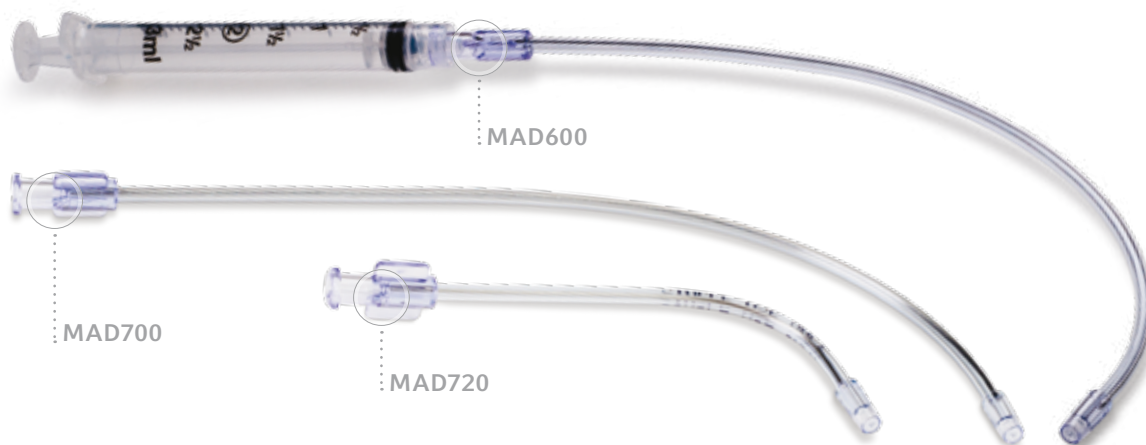


USING THE LMA® MADgic® DEVICE

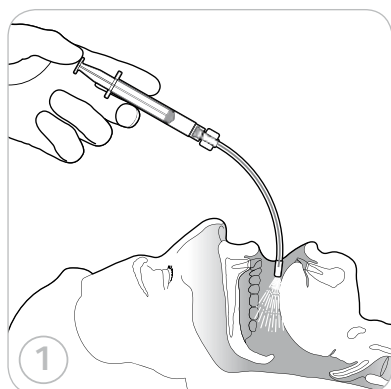

STEP 1: Atomize topical anesthetic into nose/mouth/ throat using the MADgic® Device.

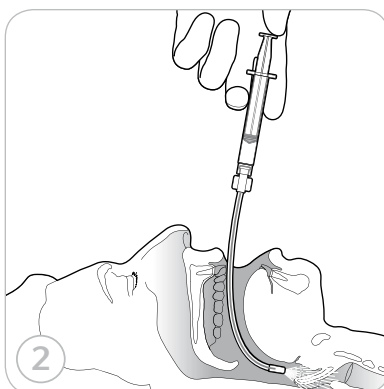

STEP 2: Bend the MADgic® stylet into a 90° curve, slip past tongue and atomize additional anesthetic into glottic area.

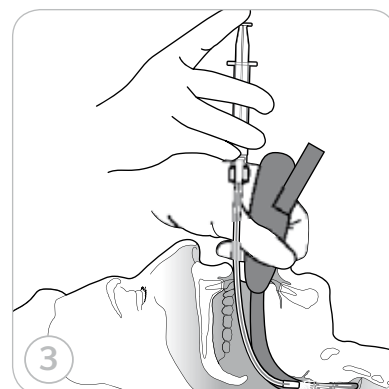

STEP 3: Position patient for intubation, atomize anesthetic onto vocal cords. Proceed to intubate as desired.

LMA® MADgic® DEVICE SPECIFICATIONS

Typical particle size	30-100 microns
System dead space	0.13 mL MAD720 / 0.25 mL MAD600 / 0.19 mL MAD700
Tip diameter	0.18 inches (4.6 mm)
Applicator length	4.5 inches (11.4 cm) MAD720 / 8.5 inches (21.6 cm) MAD600 and MAD700

LMA® MADgic® DEVICE ORDERING INFORMATION

PRODUCT DESCRIPTION	QUANTITY/BOX	CATALOG NUMBER
LMA® MADgic® Laryngo-Tracheal Mucosal Atomization Device with 3 mL Syringe	25	MAD600
LMA® MADgic® Laryngo-Tracheal Mucosal Atomization Device without 3 mL Syringe	25	MAD700
LMA® MADgic® Pediatric Laryngo-Tracheal Mucosal Atomization Device without 3 mL Syringe	25	MAD720

For use with drugs approved for intranasal and oropharyngeal delivery.

TO ORDER, CALL 1.866.246.6990 OR VISIT OUR WEBSITE WWW.LMANA.COM

Teleflex, LMA, LMA Atomization, MAD, and MADgic are trademarks or registered trademarks of Teleflex Incorporated or its affiliates.
© 2013 Teleflex Incorporated. All rights reserved. 2012-1532