

ReVel[®] Job Aid: Presets Values

Presets Values for Alarm Configuration/Limits

Table 1: Presets Values for Alarm Configuration/Limits

Controls	Infant (5-10 kg)	Pediatric (10-40 kg)	Adult (> 40 kg)
Apnea Interval	20 sec	20 sec	20 sec
High f	80 bpm	60 bpm	40 bpm
High PEEP	8 cmH ₂ O	11 cmH ₂ O	11 cmH ₂ O
High Pres	30 cmH ₂ O	30 cmH ₂ O	40 cmH ₂ O
HP Delay	Delay 0 Brth	Delay 0 Brth	Delay 0 Brth
Low FiO₂	18%	18%	18%
Low Min Vol	0.5 L	1.0 L	3.0 L
Low PEEP	1 cmH ₂ O	3 cmH ₂ O	3 cmH ₂ O
Low Pk Pres	10 cmH ₂ O	10 cmH ₂ O	10 cmH ₂ O
LPP Alarm	Control Only	Control Only	Control Only
Pulse Rate, High Alarm	"- - -" (Off)	"- - -" (Off)	"- - -" (Off)
Pulse Rate, Low Alarm	"- - -" (Off)	"- - -" (Off)	"- - -" (Off)
Safety Valve	10 cmH ₂ O	10 cmH ₂ O	10 cmH ₂ O
SBT High PEEP	8 cmH ₂ O	11 cmH ₂ O	11 cmH ₂ O
SBT Low PEEP	1 cmH ₂ O	3 cmH ₂ O	3 cmH ₂ O

NOTE:



Alarms not listed in the table above are unaffected by the Presets feature and normal ventilation will begin using the settings that were in effect the last time the ventilator was powered down. Refer to the *ReVel Operator's Manual* for detailed information concerning all alarms.

Caution: Federal Law restricts this device to sale, distribution and use by or on the order of a physician.