


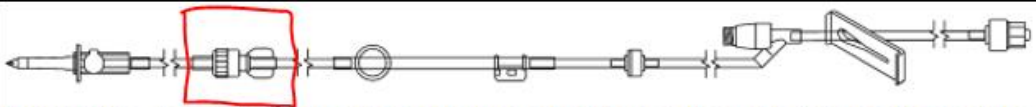

CLINICAL OPERATIONS EQUIPMENT ALERT

APRIL 9, 2019

EQUIPMENT	CME Bodyguard 121 Infusion Pump 150" IV Tubing
<p>ALERT</p>	<ul style="list-style-type: none"> We had a report of the 150" tubing separating in the middle of the tubing during an infusion (see image below). The patient was dependent on a vasopressor. There was a short delay in correcting this once it was discovered causing the patient to become hypotensive.  <p>Vented spike, female luer lock, male luer lock, ring key, Bodysset Key, anti-siphon valve, needleless Y-site, slide clamp, male luer lock/anti-siphon valve. Priming Volume 8mL, length: 150in (380 cm)</p>
<p>REQUIRED ACTIONS</p>	<ul style="list-style-type: none"> The 150" tubing is the only set of CME Bodyguard IV tubing that has a separation point in the tubing. When using this tubing it is important to make sure the connection between the upper and lower tubing is fully engaged and secure prior to initiating an infusion.
<p>COMMENTS</p>	<ul style="list-style-type: none"> Submit a communication in myPHI for any event of this nature.