



CareFusion® Learning Portal

A new way to learn about our products.

A new Web Based course is now available on the CareFusion Learning Portal for customers using the ReVel™ Ventilator.

This course will orient users to features and functions on the ReVel and explain how to use the device.

The course is aimed at clinicians who already have experience using mechanical ventilator support.

The course number is:
CLP1113 - ReVel Ventilator

To get started on the CareFusion Learning Portal

1. Navigate to <https://clp.carefusion.com>
2. Select **Register**
3. Enter your contact information, and select a user name and password
4. Select **Submit**, and watch for an email from CustomerLearning@CareFusion.com
5. Click the link to activate your account and then log in with the user name and password you selected in step 3

CareFusion ReVel™ Ventilator Lesson 1: Overview Page 10

Navigating Menu—Screen 1

Startup Menu (x3)

- 1 Same Patient
 - Intubated
 - NPPV
- 2 New Patient
 - Patient ID
 - Patient Size
- 3 Vent Check
 - Patient ID
 - Language
 - Units
 - Button Test
 - Circuit Test
 - Reset
 - FIO2 Sensor
 - About

Apnea Backup Ventilation Lesson 3: Breath Types and Modes Page 33

Example of Apnea Backup

Apnea Backup Ventilation Breath Rate	In CPAP-PS and NPPV Ventilation Modes ONLY
If the set Breath Rate is less than 12 bpm, the Apnea Backup ventilation breath rate is the highest rate allowed by control limiting, up to 12 bpm.	If the set Breath Rate is equal to or higher than 12 bpm, then the Apnea Backup ventilation breath rate will be the set Breath Rate.

Roll over the icon to learn more.

Access the CareFusion Learning Portal at:

<https://clp.carefusion.com>

